

# ZEN™ Free & Clear

wireless footswitch

## Zen™ Wireless Footswitch

The Zen™ Wireless Footswitch from ConMed Linvatec offers reliable wireless operation of Linvatec powered instruments in a convenient, ergonomic design, which allows for flexibility in the operating room. The Footswitch and Adaptor communicate using a highly reliable low power connection. Lower power requirements mean engaging the battery less often.

- No connection cable, reduces the fuss encountered with most tethered footswitches
- Low power consumption, allows for prolonged battery life and less maintenance
- Rugged construction with an ergonomic design and a conveniently integrated carrying handle
- 3 Separate pedals provide individual command options for Forward, Oscillate, and Reverse with one easy step
- Simple set-up process



# ZEN<sup>®</sup> Wireless Footswitch

## Zen<sup>™</sup> Wireless Footswitch

The Zen<sup>™</sup> Wireless Footswitch from ConMed Linvatec offers reliable wireless operation of Linvatec powered instruments in a convenient, ergonomic design, which allows for flexibility in the operating room. The Footswitch and Adaptor communicate using a highly reliable low power connection. Lower power requirements mean engaging the battery less often.

The Zen<sup>™</sup> Footswitch communicates directly with an adaptor that allows remote operation of ConMed Linvatec's E9000, E9000G, E9000J, PowerPro<sup>®</sup>, PRO2000, PRO2000I, and Advantage D3000, D3000I and 3000J handpieces.

## Ordering Information

Product Code	Description
W1000	Wireless Footswitch
W1100	Wireless Adaptor

## Quick and Easy Set up!

1. Turn on the console.
2. Insert W1100 Adaptor into Footswitch Connector on the console.
3. Press the blue button on the Adaptor. The LED on the adaptor will illuminate amber and begin to blink.
4. Depress any of the Footswitch Pedals. LED on the footswitch begins to flash amber while the two devices sync up.
5. When both lights glow green, the system is ready for use.



Utilizes two regular "D" Alkaline Batteries



Made in the USA

CONMED CORPORATION PRODUCT AREAS:

ARTHROSCOPY • ELECTROSURGERY • ENDOSCOPY • ENDOSURGERY • GASTROENTEROLOGY • INTEGRATED SYSTEMS • PATIENT CARE • POWERED INSTRUMENTS • PULMONOLOGY



11311 Concept Boulevard  
Largo, FL 33773-4908  
(727) 392-6464  
Customer Service: 1-800-237-0169  
FAX: (727) 399-5256  
International FAX: +1 (727) 397-4540  
email: [customer\\_service@linvatec.com](mailto:customer_service@linvatec.com)  
[www.linvatec.com](http://www.linvatec.com)

© 2009, Linvatec Corporation, a subsidiary of CONMED Corporation, 04/2009, CBR 0039