

The Most Versatile Suture Passer
used in soft tissue repairs

Spectrum® MVP™

THE MOST VERSATILE SUTURE PASSER

- For use in instability or rotator cuff repairs
- Unique "Single-pass" instrument
- Pass and Retrieve with one device
- Ergonomic handle
- Quick and Easy to use
- Patent pending

Spectrum[®] MVP[™] Suture Passer




True innovation returns with the release of the Spectrum MVP from ConMed Linvatec's Sports Medicine Team. This suture passer (patent pending) will offer surgeons a new perspective on the ease of suture passing. Over the past two decades the Spectrum brand has provided many solutions to suture passing, but the latest edition will improve upon all the technology we have developed. This new system will offer another option in the long tradition of the Spectrum line of products. The "Gold Standard" in suture passing is about to change.

The Spectrum MVP was designed to be the Most Versatile suture passer on the market. The world-class engineers of ConMed Linvatec designed an instrument that will provide simplified suture passing options for years to come.




Instability Repair


MVP passed through the labrum with the loop deployed and the suture clip approaching

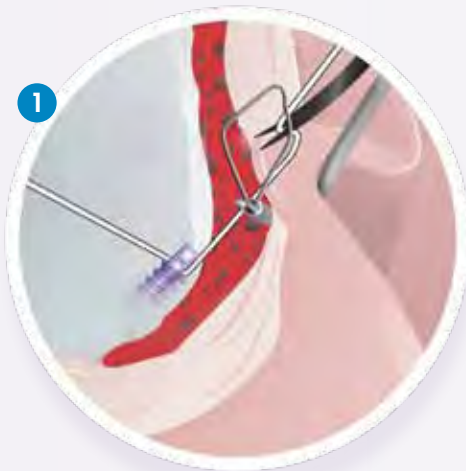


MVP passed through the labrum with the loop collapsed and the suture clip through the loop

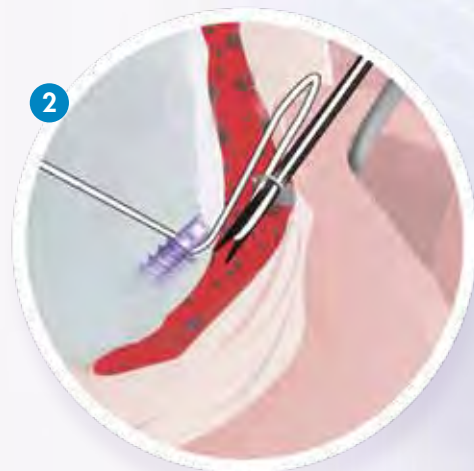


MVP passed through the labrum with the loop collapsed on the suture

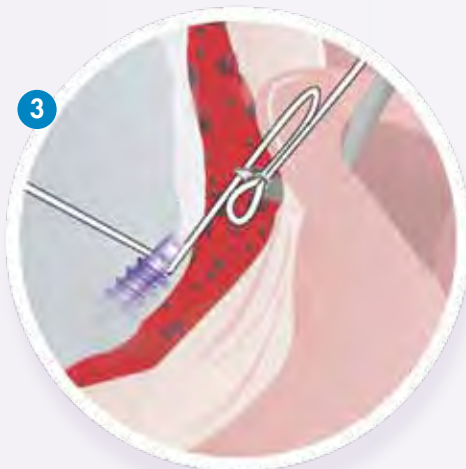


MVP pulling the suture through the labrum

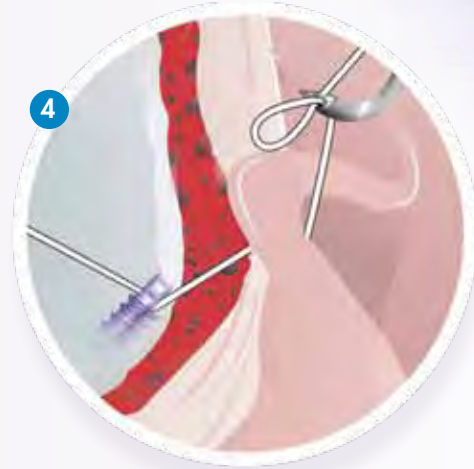


MVP passed through the rotator cuff with the loop deployed and the suture clip approaching

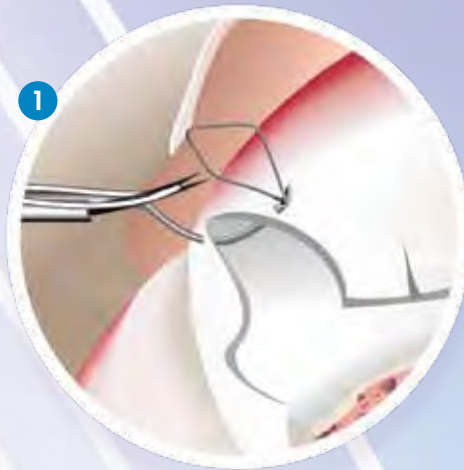


MVP passed through the rotator cuff with the loop collapsed and the suture clip through the loop

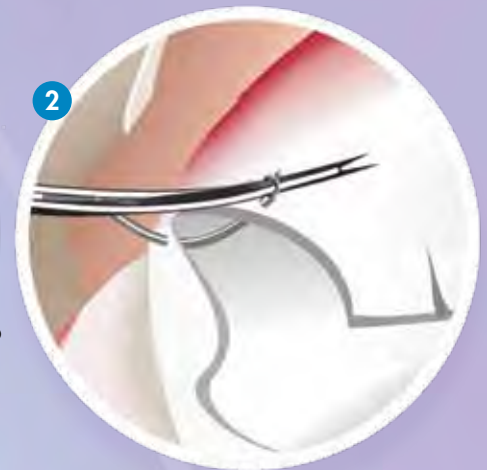

MVP passed through the rotator cuff with the loop collapsed on the suture

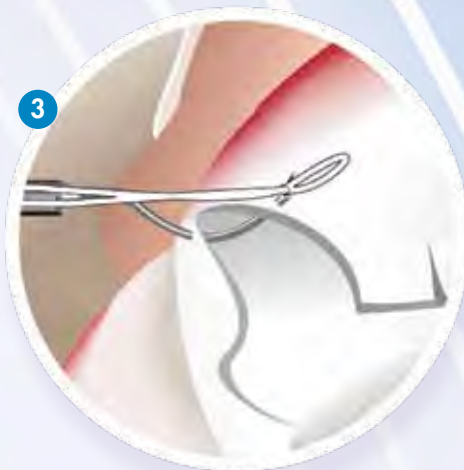


MVP pulling the suture through the rotator cuff

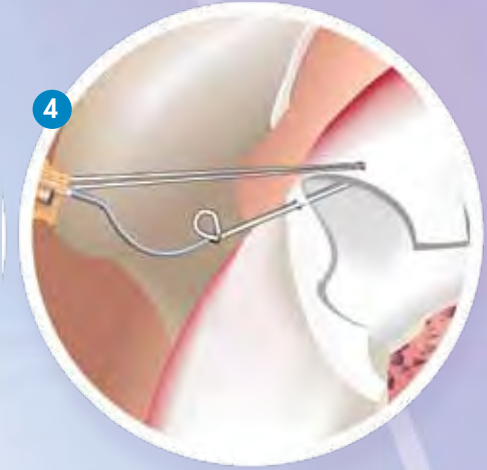

Basic Technique




Create a small loop at the tip of the post strand of suture and place it in the Suture Guide Slot.

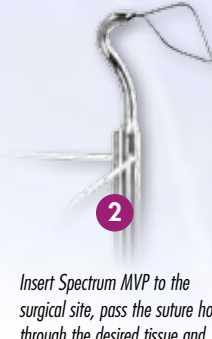


Insert Spectrum MVP to the surgical site, pass the suture hook through the desired tissue and deploy the Suture Loop by sliding the **BLUE** switch forward to the lock position.



Pass the suture through the suture loop by sliding the **GREY** Suture Guide switch forward.

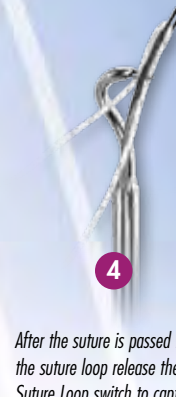

After the suture is passed through the suture loop release the **BLUE** Suture Loop switch to capture the suture. The spring assisted switch will close on the suture.

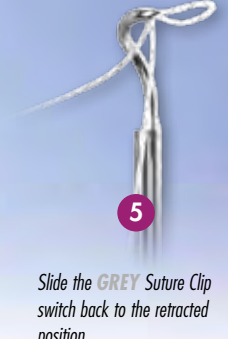

Slide the **GREY** Suture Clip switch back to the retracted position.

6


With your finger/thumb on the **BLUE** switch retract the suture hook through the tissue and the suture is passed.

**Technique Tip – During arthroscopic procedure keep slight tension on the suture and hold extra suture to the side in the cannula*

Spectrum® MVP™ Suture Passer



Description	Cat. No.
Spectrum MVP, Medium Crescent Hook, Disposable Packaged Sterile, 3/box	C6400
Spectrum MVP, 45° Right Hook, Disposable..... Packaged Sterile, 3/box	C6410
Spectrum MVP, 45° Left Hook, Disposable Packaged Sterile, 3/box	C6420
Spectrum MVP, Medium Crescent Hook, Extended length, Disposable..... Packaged Sterile, 3/box	C6400EL
Spectrum MVP, 45° Right Hook, Extended length, Disposable Packaged Sterile, 3/box	C6410EL
Spectrum MVP, 45° Left Hook, Extended length, Disposable Packaged Sterile, 3/box	C6420EL
Accessories	
Crochet Hook	C6105
Loop Handle Knot Pusher	C6112
Katana™ High Strength Suture Cutter, 4mm Diameter, Straight	GU1009
Grasping Forceps, 3.4mm Diameter, Straight with Ratchet	11.1001
Suture Retrieval Forceps, 3.4mm Diameter	16.1018
Liberator™ Knife	25.50014
Rasp Liberator™ Knife	25.50016
Suture Saver™ Kit (5 kits/box)	C6180
#2 Hi-Fi™ High Strength Suture, Double-pack	H5100
12/box 40", single white strand and single strand of white with blue stripe, 1/2" tapered needle	
#2 Hi-Fi™ High Strength Suture, Single-pack.....	H5000
12/box 40" single white strand, 1/2" circle, tapered needle	

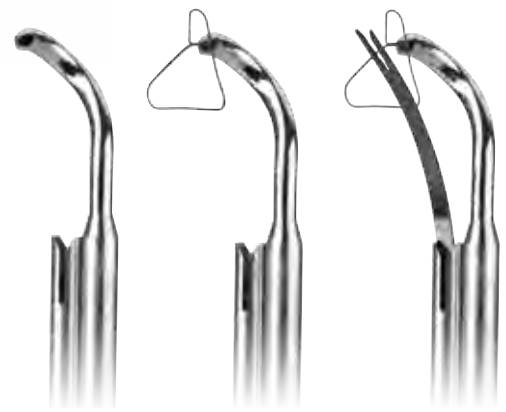

CONMED CORPORATION PRODUCT AREAS:
 ARTHROSCOPY • ELECTROSURGERY • ENDOSCOPY • ENDOSURGERY • GASTROENTEROLOGY • INTEGRATED SYSTEMS • PATIENT CARE • POWERED INSTRUMENTS • PULMONOLOGY

CONMED
 LINVATEC
 ARTHROSCOPY • ENDOSCOPY • POWERED INSTRUMENTS
 11311 Concept Boulevard
 Largo, FL 33773-4908
 (727) 392-6464
 Customer Service: 1-800-237-0169
 FAX: (727) 399-5256
 International FAX: +1 (727) 397-4540
 email: customer_service@linvatec.com
 www.linvatec.com